XKM 3000 L Med

de	Montageanweisung Kommunikationsmodul
da	Monteringsanvisning kommunikationsmodul
en	Fitting instructions Communication module
es	Instrucciones de montaje Módulo de comunicación
fi	Asennusohje Tiedonsiirtomoduuli
fr	Notice de montage Module de communication
hr	Upute za ugradnju Komunikacijski modul
it	Istruzioni di montaggio Modulo di comunicazione
nl	Montagehandleiding communicatiemodule
no	Monteringsveiledning Kommunikasjonsmodul
pt	Instruções de montagem Módulo de comunicação
sl	Navodila za montažo Komunikacijski modul
sv	Monteringsanvisning Kommunikationsmodul

de	4
da	19
en	34
es	49
fi	64
fr	79
hr	94
it	109
nl	124
no	139
pt	154
sl	169
sv	184

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Notes about these instructions	34
Intended use	35
Intended use	35
Queries and technical problems	35
Items supplied	36
Items supplied	36
Optional accessories	36
Guide to the appliance	37
Overview	37
Switching on and off	37
Control LEDs	38
Reset button	38
Reset operational faults	38
Reset factory settings	38
Warning and Safety instructions	39
Fitting and installation	40
Module slot	40
Installation	41
Configuring the interface	42
Configuring the network interface	42
Opening the menu	42
Check the connection status	43
Selecting settings	43
Accept configuration	44
Fault 559	45
Web interface	45
Technical data	46
Caring for the environment	47
Disposal of the packing material	47

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Intended use

With Miele washer-disinfectors, cleaning processes can be documented. To enable this, Miele washer-disinfectors are equipped with a module slot on the back to take a Miele communication module.

The XKM 3000 L Med communication module enables the installation of an Ethernet interface.

For connection to a WiFi network, the Miele communication module can be connected to a wireless access point provided by you, using a cable.

Only use terminal devices (PCs etc.) which are approved according to EN/IEC 60950.

For more information on suitable software for process documentation please contact Miele.

Configuration of the interface must only be carried out by suitably qualified and competent personnel.

The communication module can be used in the following washer-disinfectors:


- PG 8562
- PG 8581
- PG 8582
- PG 8582 CD
- PG 8583
- PG 8583 CD
- PG 8591
- PG 8592
- PG 8593

The communication module is generally referred to as the module in these Operating instructions.

Queries and technical problems

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to www.miele-professional.com.

Items supplied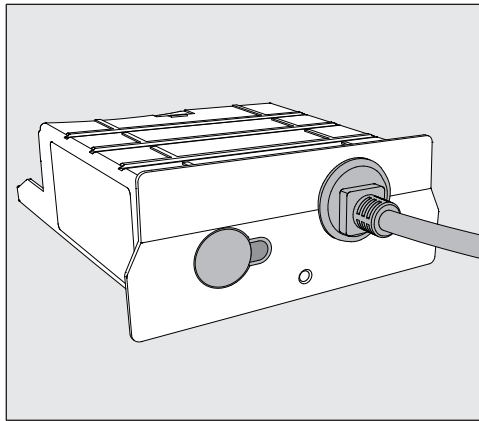


- Communication module XKM 3000 L Med

Optional accessories

The following and other accessories are available to order from Miele:

- Segosoft Miele Edition, Process documentation software for use in a PC or with a network connection.
- Additional licences for Segosoft Miele Edition
- Segosoft Miele Edition: USB version, Process documentation software for data transfer using a USB stick
- Various network cables

Overview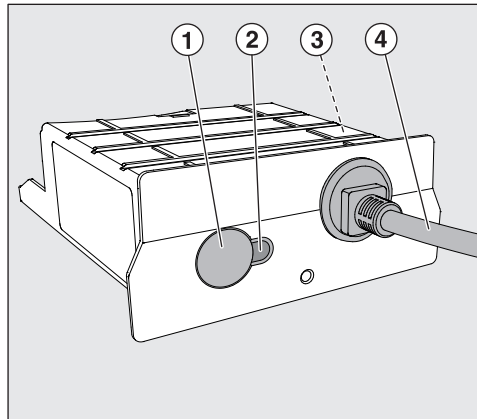

- ① Reset button
- ② Indicator LED,
Indicates ready-for-use status after installation and active communication during operation
- ③ Data plate (located on the side)
- ④ Data cable

Switching on and off

The module does not have a separate On/Off button. It is switched on and off in conjunction with the washer-disinfector.

Every time it is switched on the module takes about 90 seconds to initialise itself. Until this has happened data cannot be transferred and interface settings cannot be changed.

Control LEDs

The control LEDs (light emitting diodes) give status information about the communication module.

LED	Status
LIT UP GREEN	The module is ready for operation.
FLASHES GREEN	The module is receiving or transmitting data.
LIT UP YELLOW	The module is receiving network data, the IP address set is, however, not recognised. The interface may need to be reconfigured.
FLASHES YELLOW	Network connection being made.
LIT UP RED	Resetting in progress.
FLASHES RED	The module is initialising. This process can take up to 90 seconds.
FLASHES alternately YELLOW and RED or GREEN and RED	The module is in service mode and is not ready for use. Switch the washer-disinfector off and then back on again to come out of service mode.

Reset button

Reset operational faults

If the module experiences an operational fault during operation, the reset button can be used to get it operational again.

- To do so press the reset button for approx. 1-2 seconds.

 Do not press it for any longer than this as this would reset the module to its factory default settings and you would then have to reconfigure the interface.

The module will then restart (reset). This process can take up to 90 seconds. The LED next to the button will light up red during this process.

Reset factory settings

If you cannot get the module operational again with the reset button you will have to reset the module to its factory default settings.

 If the factory default settings are reset the interface will have to be reconfigured.

- Press and hold the reset button for approx. 5-6 seconds.

The LED will start to flash red. Resetting to default settings takes at least 90 seconds.

To avoid the risk of accidents and damage to this communication module please read these instructions carefully before using it for the first time.
Keep these instructions in a safe place.

 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.


- ▶ This communication module must only be used for applications specified in the operating instructions. Changes or modifications to it and incorrect use of it are not permitted as this can be dangerous. Miele cannot be held liable for damage resulting from incorrect use or handling.
- ▶ The communication module must only be installed, commissioned and serviced by Miele service or by a suitably qualified person. Repairs are only to be undertaken by Miele Service. Repair work by unqualified persons could be dangerous.
- ▶ Check the communication module carefully before installation for any visible damage. A damaged module could be dangerous and must not be installed.
- ▶ Before installing the communication module compare the connection data (voltage and frequency) on the data plate with that on the data plate of your washer-disinfector. If the data does not match, the module could sustain damage. If in any doubt, consult an electrician.
- ▶ The communication module housing must not be opened on any account.
Touching live electrical components or altering the electrical or mechanical components is dangerous and could cause the module to malfunction.

Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

Module slot

The module is inserted into the module slot on the back of the washer-disinfector.

- If the washer-disinfector is installed in a niche:
 - Pull the washer-disinfector forwards out of the niche.
- If the washer-disinfector is installed under a worktop:
 - Open the door.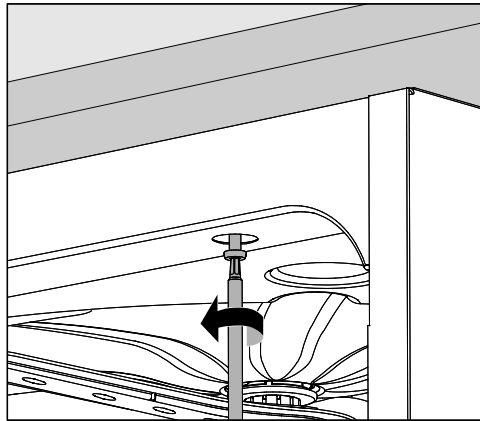


- Unscrew the fixing screws.
- Pull the washer-disinfector out from under the worktop.

Installation

⚠ Before installing the communication module compare the connection data (voltage and frequency) on the data plate with that on the data plate of your washer-disinfector. If the data does not match, the module could sustain damage. If in any doubt, consult an electrician.

- Disconnect the washer-disinfector from the mains electricity supply.
- Remove any stickers or protective caps as necessary from the slot aperture.

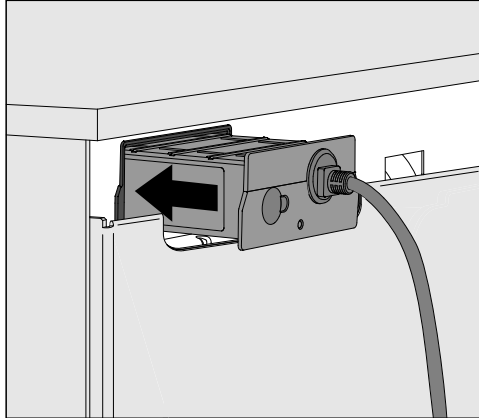- Push the module into the module slot. Gently push the module until it clicks into place.
- Establish a cable connection between the module and your terminal device.
Make sure that the cable is laid correctly and that it is not kinked or trapped anywhere. It must be protected from sharp edges and not subjected to tension.

Only use terminal devices (PC, printers etc.) which comply with EN/IEC 60950.

- Connect the washer-disinfector to the electrical supply.
The module is supplied with electricity via the washer-disinfector. An additional electrical connection is not necessary.

Configuring the network interface

⚠ The interface must only be configured by suitably qualified and competent persons.

Opening the menu

- Switch the washer-disinfector on using the ⏻ button.

You must wait at least 90 seconds before starting to configure the interface.

- Open the menu as follows:

☰ button

▶ Further settings

▶ Interface

- Select the Ethernet interface and confirm your choice with OK.

The following options are available:

– Module status

Connection status displayed.


– DHCP

The Ethernet interface can either be implemented via a Dynamic Host Configuration Protocol (DHCP) or by setting one of the following parameters:

- IP address
- Subnet mask
- Standard gateway
- DNS server automatic
- DNS server 1
- DNS server 2
- Port type
- Port

Check the connection status

- Select the Module status option and confirm your choice with **OK**.

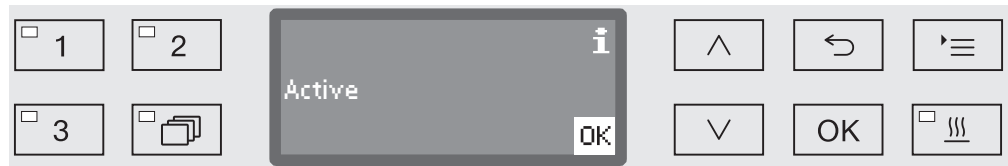One of the following messages may appear in the display:

- Active
The module is ready for operation.
- Initialising, module not yet active
The module has not finished initialising. This process takes approx. 90 seconds.
- Inactive
A module has not be recognised.
Check the module installation. Switch the washer-disinfector off and then back on again.
If the message appears in the display again call Miele Service.

Selecting settings

Document all settings in case of a subsequent Service call requirement.

For manual entries DHCP must first be deactivated.

- Select the DHCP option and confirm your choice with **OK**.

- Select the Off option and confirm your choice with **OK**.

The following settings are then available:

- IP address
 - Subnet mask
 - Standard gateway
 - DNS server automatic
 - DNS server 1
 - DNS server 2
 - Port type
 - Port
- Select an option and confirm your choice with **OK**. You can then carry out settings as shown in the display.

en - Configuring the interface

The following display is an example of entering IP address, Subnet mask etc.:

The current setting is shown in the middle line. This can be altered using the options shown in the bottom line. The top line shows the selected option. The following input options are available:

- Numbers from 0 to 9.
- Decimal point ".".
- Use the  symbol to delete the last position.
- Use the  symbol to save your setting. The display will then return to the main menu.
- The  symbol in the display or the  button end the process without saving the setting. The display reverts to the initial menu.

Always enter the IP address, the Subnet mask etc. fully. For instance if the IP address is 192.168.1.23 you must enter 192.168.001.023.

- Use the arrow buttons  (right) and  (left) to move the cursor to the option you require.
- Confirm each entry with **OK**.
- Save your settings by selecting the  symbol in the display and then confirm your choice with the **OK** button.

Accept configuration

The interface configuration is only effective after the washer-disinfector has been restarted.

- Wait approx. 10 seconds before switching the washer-disinfector off for a restart.
- Switch the washer-disinfector off and then back on again with the  button.
- Wait approx. 90 seconds for the module to initialise.
- Then carry out a functional test of the interface.

Fault 559

If fault message Fault 559 appears when the machine is started or when a programme is aborted, the RS232 interface is active and has to be deactivated.

- Open the menu for configuring the RS232 interface and then select Further settings / Interface.
- Select the Ethernet option and confirm your choice with OK.

The RS232 interface has now been deactivated. Wait approx. 90 seconds for the module to be operational.

Web interface

The XKM 3000 L Med module has its own web interface, which is used for accessing general data about the washer-disinfector, changing network settings and loading updates to the module.

The interface can be accessed from any workstation in your local network. To do so there must be a configured serial interface in the machine's controls, a current web browser and a network connection to the machine. The output language is English.

- Enter the IP address for your washer-disinfector in the address line in your browser.

The web interface for the module will continue to be shown in your browser. Data can take up to 60 seconds to transfer from the washer-disinfector to the web interface.

The web interface is arranged as follows:

– **Appliance information**

This page contains general information about the washer-disinfector, e.g. model number (*Model number*), serial number (*Serial number*) or MAC Address (*MAC address*).

– **System settings**

Data about the interface, such as the IP address (*IP address*) or the subnet mask (*Subnetmask*) can be found on this page. This data can be changed. It is password protected.

– **Password settings**

You can change your password on this page.

– **Firmware update**

If necessary, updates to the module's software can be loaded from this page. This function is password protected.

The default password is available from the Service department.

If this password has been changed it will revert to the factory default one if the module is reset.

en - Technical data

Voltage, Connected load, Fuse rating	See data plate
Mains cable	Approx. 1.5 m
Transfer speed	10/100 Mbit/s
Temperature during operation	Up to max. 80 °C
Test certificates awarded	UL Recognised Component 
CE mark	Low Voltage Directive 2006/95/EC, EMC Directive 2004/108/EC, RoHS Directive 2011/65/EU
Manufacturer's address	Miele & Cie. KG, Carl-Miele-Straße 29, 33332 Gütersloh, Germany


Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

Disposing of your old appliance

Electrical and electronic appliances often contain valuable materials. They also contain specific materials, compounds and components, which were essential for their correct function and safety. These could be hazardous to human health and to the environment if disposed of with your domestic waste or if handled incorrectly. Please do not, therefore, dispose of your old appliance with your household waste.

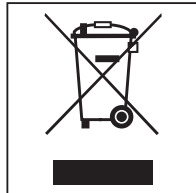Please dispose of it at your local community waste collection / recycling centre for electrical and electronic appliances, or contact your dealer or Miele for advice. You are also responsible (by law, depending on country) for deleting any personal data that may be stored on the appliance being disposed of. Please ensure that your old appliance poses no risk to children while being stored prior to disposal.

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