Miele DataDiary

de	Kurzanleitung App
da	Kort vejledning App
en	Brief instructions for app
fr	Notice Application
it	Istruzioni brevi App
es	Guía rápida de la App
nl	Verkorte gebruiksaanwijzing app
pl	Skrócona instrukcja obsługi Aplikacja
pt	Guia de iniciação rápida App
sv	Snabbinstruktion App

de	4
da	27
en	50
fr	73
it	96
es	119
nl	142
pl	165
pt	188
sv	211

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Protokolle freigeben

The screenshot shows a mobile application interface for creating a release protocol. On the left, there is a list of process parameters: 'Drying block', 'Spray arm monitoring OK', 'Spray pressure OK', 'A0 value OK', and 'Conductivity OK'. Below this, it states 'Programme finished' and 'Process parameters achieved' with a timestamp '08:15:44 FINISH'. On the right, a question asks 'Kann das Chargenprotokoll erstellt werden?' (Can the batch protocol be created?). Below this question are three checkboxes: 'Prozess-Parameter erfüllt' (checked), 'Visuelle Chargen-Kontrolle in Ordnung' (checked), and 'Protokoll in Ordnung' (unchecked). At the bottom right, there are three buttons: 'Störungs-Protokoll erstellen' (Create Fault Protocol), 'Freigabe-Protokoll erstellen' (Create Release Protocol), and 'Später bearbeiten' (Edit Later).

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The screenshot shows the authentication screen of the mobile application. It asks 'Bitte authentifizieren Sie sich' (Please authenticate yourself). There are three input fields: 'Nutzer auswählen' (Select user), 'Passwort' (Password), and 'Passwort anzeigen' (Show password). Below these fields is an NFC tag icon and the text 'Authentifizieren via NFC-Tag' (Authenticate via NFC tag). At the bottom, there are three buttons: 'Zurück' (Back), 'Bestätigen' (Confirm), and 'Später bearbeiten' (Edit Later).

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
The screenshot shows the same 'Rapport' screen as before, but the dialog box is now titled 'Identificer dig'. It contains three input fields: 'Vælg bruger' with a dropdown arrow, 'Kode', and 'Vis kode' with a toggle switch. Below these fields is an icon of an NFC tag and the text 'Identifikation med NFC-brik'. At the bottom of the dialog are two buttons: 'Tilbage' and 'Bekræft'. Below the dialog is a button labeled 'Vent med ændringer'.

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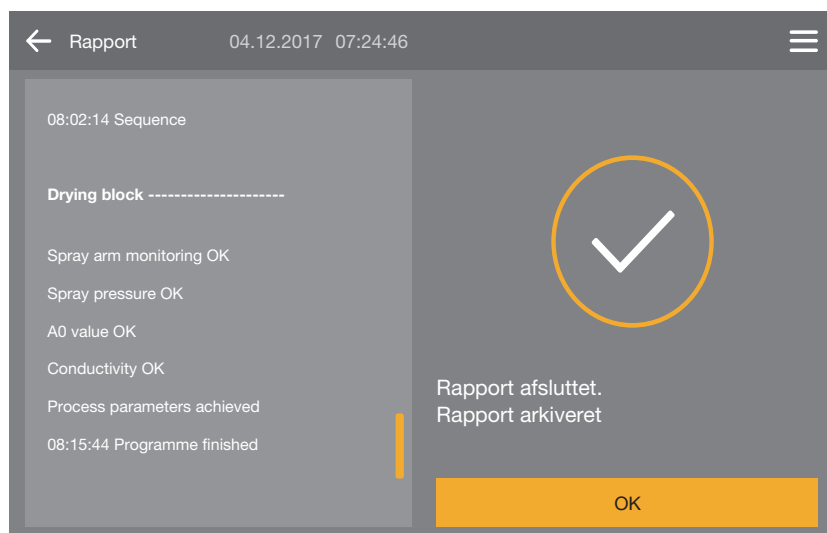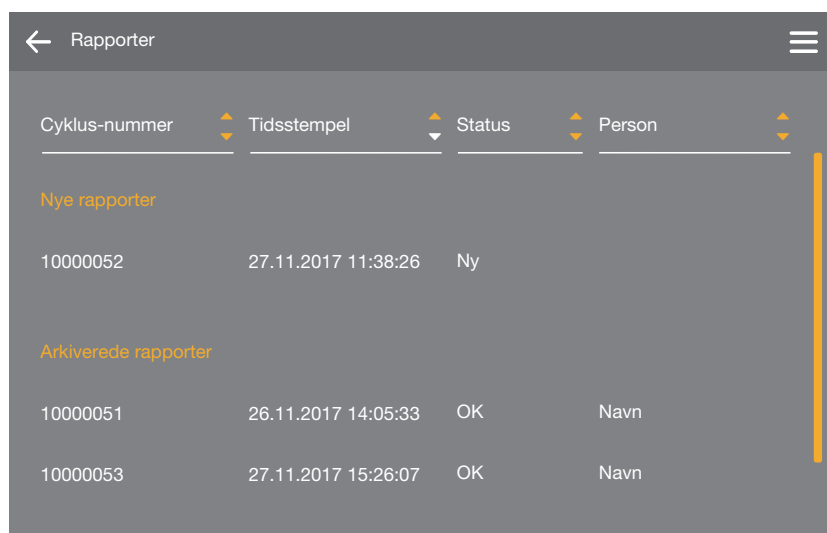
This screenshot is similar to the previous one, showing the same report data on the left. On the right, the title is 'Næste trin?'. It contains three radio buttons: 'Gentag vask/desinfektion' (selected), 'Afvis cyklus', and an empty radio button. At the bottom right, there are two buttons: 'Tilbage' (disabled) and 'Bekræft' (active). Below these buttons is a link that says 'Vent med ændringer'.

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This screenshot shows the same report data on the left. On the right, the title is 'Identificer dig'. It contains a 'Vælg bruger' dropdown menu, a 'Kode' input field, and a 'Vis kode' toggle switch. Below these fields is an NFC icon and the text 'Identifikation med NFC-brik'. At the bottom right, there are two buttons: 'Tilbage' (disabled) and 'Bekræft' (active). Below these buttons is a link that says 'Vent med ændringer'.

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
After installation	50
Creating an administrator	50
Storing the NFC tag for the administrator	51
Storing	51
Operation	52
Starting the app	52
Menu	53
Opening the menu	53
Menu options	53
Exiting	53
Live report	54
Live report	54
View on the screen	55
Switching the view	55
Reports	56
Reports	56
Retrieving reports	56
Editing incomplete reports	57
Checking reports	58
Approving reports	59
Cancelling reports	61
Aborting editing	62
Archiving reports	63
Retrospective output of reports	63
Administration	64
Administration	64
Opening the Administration menu	64
Language	65
Users	66
WiFi configuration	68
XKM FW update	69
Information	70
Information	70
Saving energy	71
Saving energy	71

Creating an administrator

- Start the Miele DataDiary app by tapping the app icon.

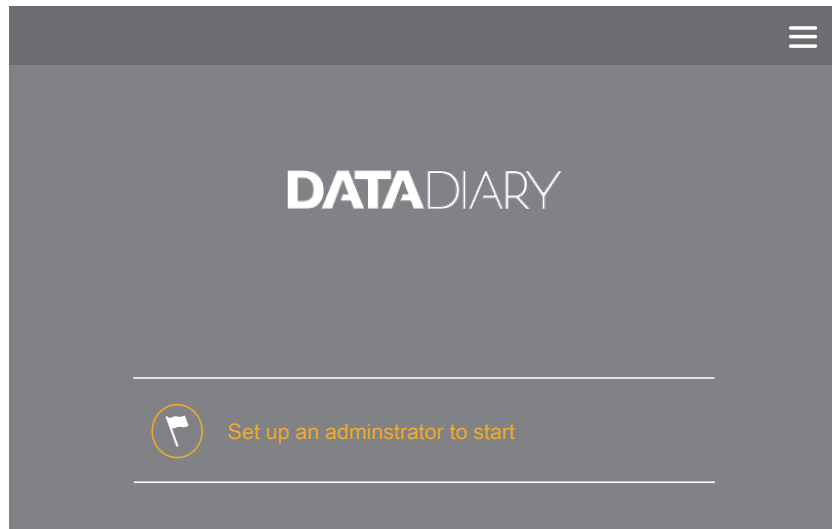When you start the app for the first time, you will be prompted to create an administrator.

- Tap the button.

The screenshot shows the 'Edit administrator account' form within the app. The title bar at the top has a back arrow and the word 'User' on the left, and a hamburger menu on the right. On the left side of the form is a dark grey sidebar with a list: 'First name' and 'Name' are grouped together, and 'Position' is below them. The main area of the form is titled 'Edit administrator account' and contains several input fields: 'First name', 'Name', 'Position', 'Password', 'Retype password', and 'Security question' (with the subtext 'What is your mother's first name?'). At the bottom of the form are two buttons: 'Cancel' in a grey box and 'Save administrator' in a yellow box.

- Complete the First name, Name and Position fields, all of which are mandatory. When you want to complete a field, start by tapping it.
- Enter a Password and answer the Security question.

Storing the NFC tag for the administrator

You then have the option of storing a chip card with an NFC (near-field communication) tag for the administrator.

If someone logs in using an NFC card, there is no need for them to enter their user name or password.

To use NFC cards, you need a tablet that supports NFC. 3 NFC cards are supplied with the XKM module. You can obtain additional cards from Miele.

- Hold the NFC card against the back of the tablet. In most cases, the NFC module that reads the NFC card is located at the edge. However, the exact position of the NFC antenna will vary from one tablet model to another.

If the message NFC tag saved: xxxxxxxx appears, where xxxxxxxx represents the ID number of the NFC card, this means that the card has been successfully assigned to the user account.

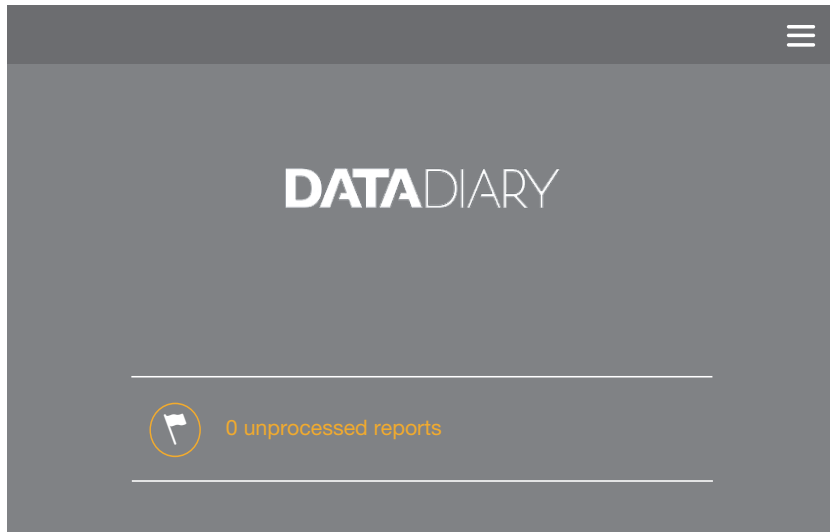
Use **Reset** to remove the NFC card assignment.

Storing

- Tap the **Save administrator** button.
- To complete the process, you need to close the app and restart it.

Starting the app

- Start the Miele DataDiary app by tapping the app icon.


The flag icon  on the start screen indicates that there are tasks which are not yet complete (editing reports, for example).

- Tap the button if you want to edit the tasks directly.
- Or
- Open the app menu.

Menu

Opening the menu

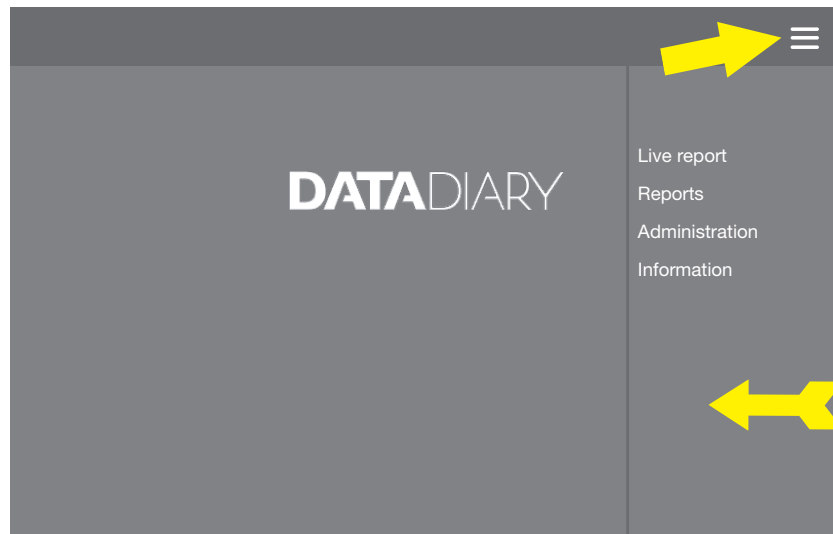The app is operated using the menu shown below. You can open the menu in the following ways:

- Tap the icon in the top right-hand corner.

Or

- Swipe from the right-hand edge of the screen towards the centre.

Menu options

The menu contains the following options:

- Live report

Here, you can follow the progress of the batch reports on the display while the current process is running. This requires an active WiFi connection to the XKM module.

- Reports

Here, you can view and edit stored reports.

- Administration

This menu option allows you to make basic settings, such as creating new users or setting up a WiFi connection.

- Information

Here, you can view information about the app, such as the IP address, the app version or the legal notice.

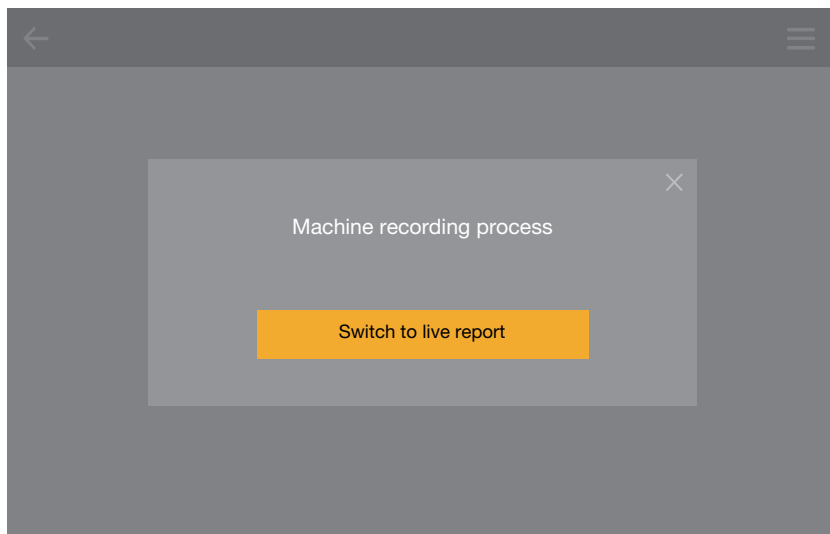
Exiting

Use the arrow icon <— at the top left to exit the menu or return to the next-highest menu level.

Live report


Here, you can follow the progress of the batch reports on the display while the current process is running. This requires an active WiFi connection between the XKM module in the machine and the tablet on which the Miele DataDiary app is running.

- Start the Miele DataDiary app by tapping the app icon.
- Switch the Miele machine on. The XKM module needs around 30 seconds to establish a connection to the tablet.
- Load the machine and select a programme.
- Start the programme on the machine.

As soon as the app starts receiving data from the machine, the view will switch to the live report view or you will be asked if you want to switch to it.

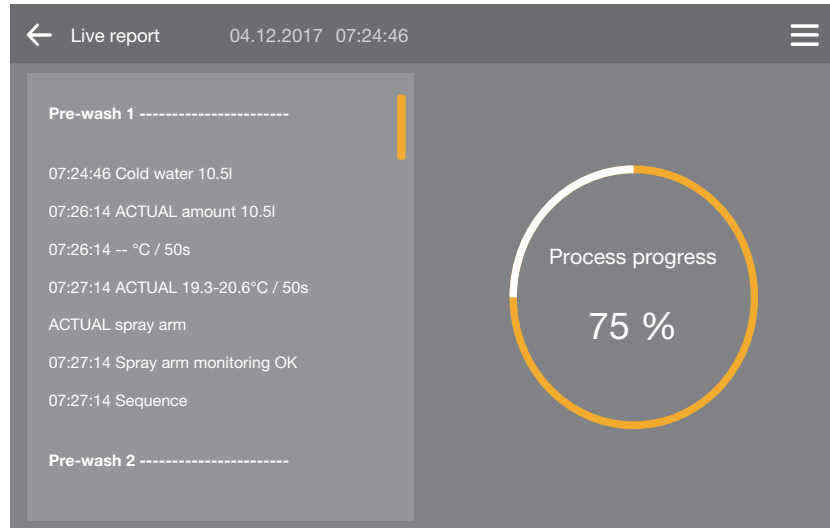View on the screen

In the live report view, the screen is divided into two:

- The left-hand side shows the individual process steps in chronologically descending order.
- The right-hand side shows the progress of the process.

Switching the view

At any time, you can use the arrow icon $\leftarrow$ at the top left-hand side to switch from the live view to the start screen or open the app menu. This will not interrupt the data transmission. You can continue to work with the app and carry out tasks such as checking unedited reports (except for the live report).

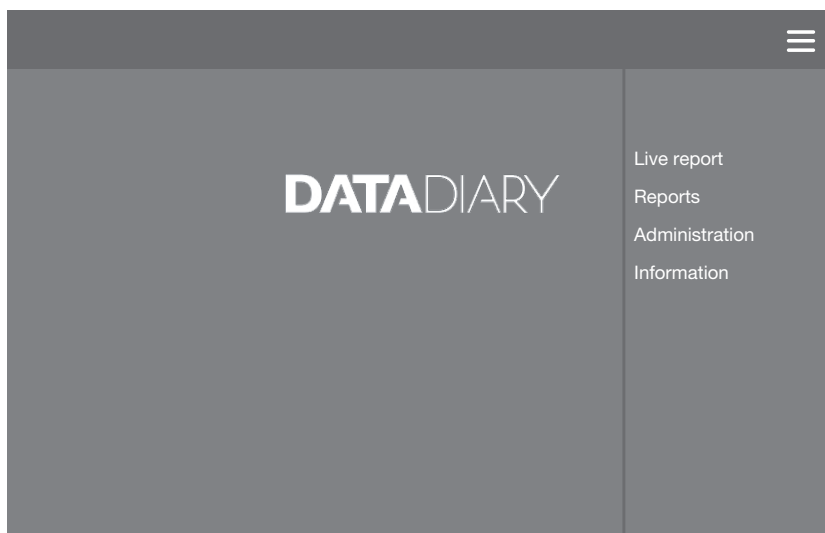
The Live report option allows you to return to the live view.

 To avoid aborting the recording process, do not close the app or interrupt the WiFi connection.


Reports

Retrieving reports Reports are stored in the report memory along with editing information.

- Open the menu by either tapping the icon in the top right-hand corner or swiping from the right-hand side of the screen towards the centre.

- Select the Reports option.

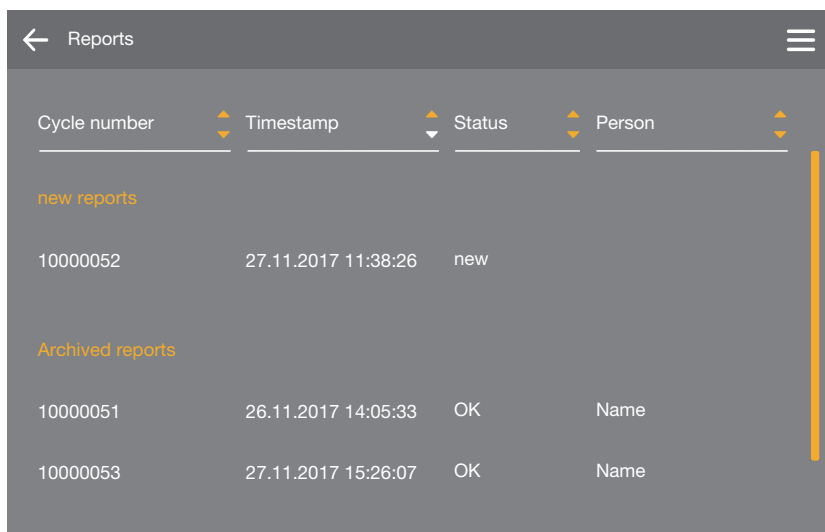The report memory is shown in the form of a list. Unedited reports appear at the top in a dedicated category. You can use the ▲ and ▼ arrows to sort the view. The type of sorting that is currently being used is indicated by a white arrow.

- Tap a line to retrieve a report.

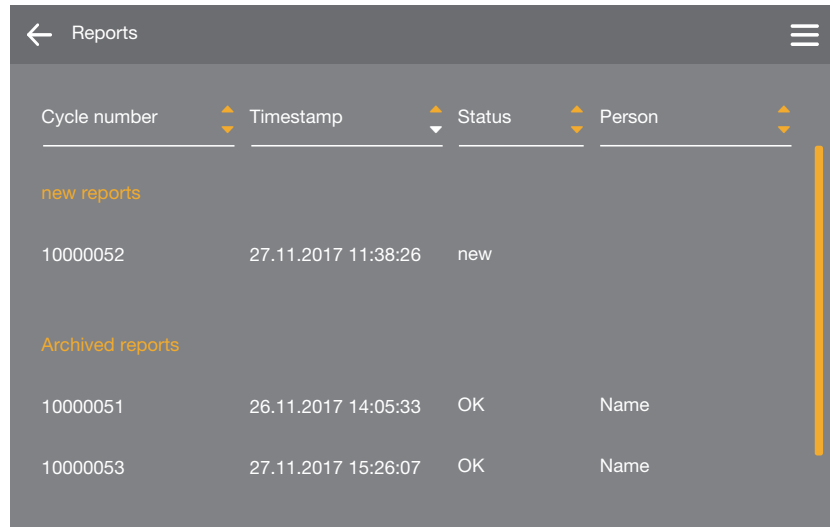
Editing incomplete reports

You should check the reports as soon as possible after the programme cycles have taken place.

- In the live report view, tap **Create report**.


Or

- Open the report list by either tapping the button for incomplete tasks or selecting the **Reports** menu option.

Cycle number	Timestamp	Status	Person
new reports			
10000052	27.11.2017 11:38:26	new	
Archived reports			
10000051	26.11.2017 14:05:33	OK	Name
10000053	27.11.2017 15:26:07	OK	Name

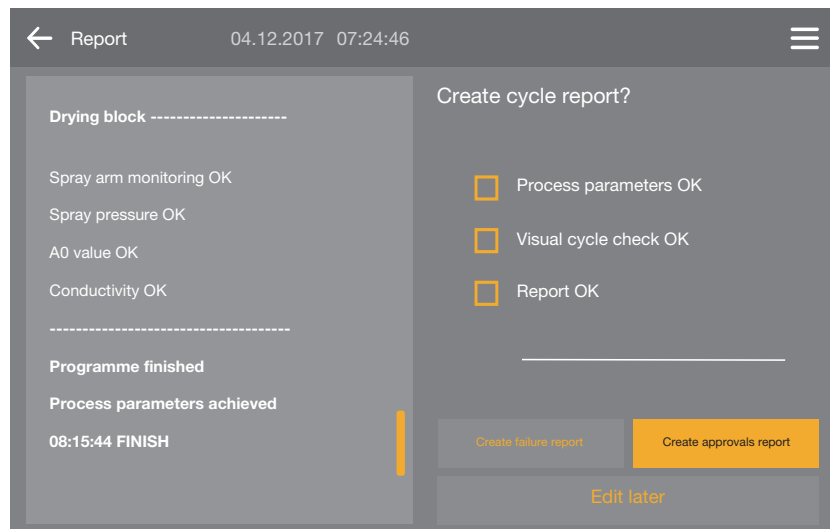- Select one of the incomplete reports.

← Report 04.12.2017 07:24:46

Drying block -----

Spray arm monitoring OK

Spray pressure OK

A0 value OK

Conductivity OK

Programme finished

Process parameters achieved

08:15:44 FINISH

Create cycle report?

☐ Process parameters OK

☐ Visual cycle check OK

☐ Report OK

Create failure report Create approvals report

Edit later

The screen for checking the report is divided into two:

- The left-hand side shows the individual process steps in chronologically descending order.
- The right-hand side documents the process of checking the report.

Checking reports The process of checking reports involves several stages:

☐ **Process parameters OK**

At the end of a programme, one of the following messages will appear on the machine display:

1. Process parameters OK
The programme has finished without any problems.
2. Process parameters not achieved
This indicates that problems occurred during reprocessing. This message usually appears together with one or more fault codes/messages.

A corresponding message is also documented in the batch report.

- On the left-hand side of the screen, scroll down to the end of the report.

Check whether the message from the report matches the message on the machine.

- If the process parameters have been complied with, tick ☒ Process parameters OK.
If the parameters have not been complied with, check the fault messages and cancel the report (see below). Do not tick ☐ Process parameters OK in this case.

☐ **Visual cycle check OK**

- Check the cleanliness; you can do this by carrying out a visual inspection, for example. If an acceptable level of cleanliness has been achieved, tick ☒ Visual cycle check OK.
If the level of cleanliness achieved is not acceptable, do not tick this box.

If you have ticked the ☒ Process parameters OK and ☒ Visual cycle check OK boxes, you can approve the report.

- Finally, check the report one more time in case there are any discrepancies – for example, the time being set to winter even though summer now applies.

☐ **Report OK**

If you do find any discrepancies, such as an incorrect time, you can still approve the report by ticking this box. Doing this will document the fact that the report and batch are considered OK despite the discrepancies.

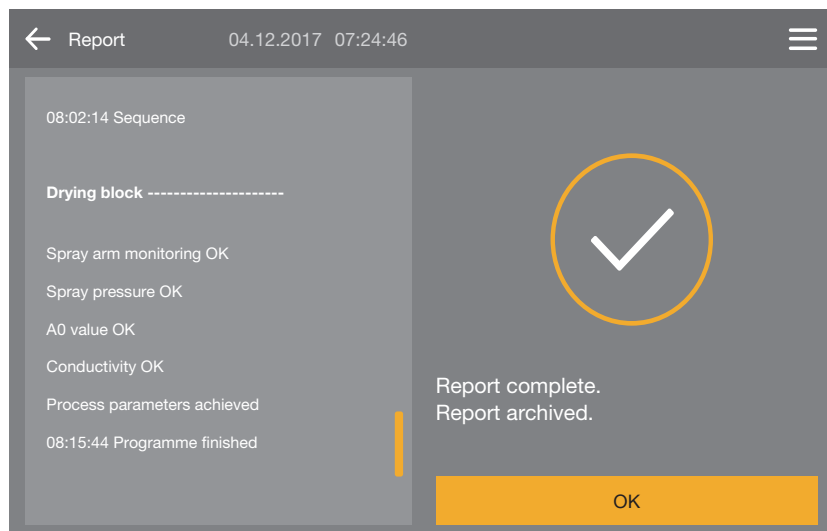
Approving reports

If the ☒ Process parameters OK and ☒ Visual cycle check OK boxes are ticked, the Create approvals report button is enabled.


- If you need to, you can enter a comment in the free text area under the boxes.
- Press the Create approvals report button to approve the report.

You will then be prompted to authenticate yourself.

- Log in with your user name and password.
- Or
- Log in using your NFC card by holding it against the NFC sensor in your tablet. The sensor is usually located on the back of the tablet.

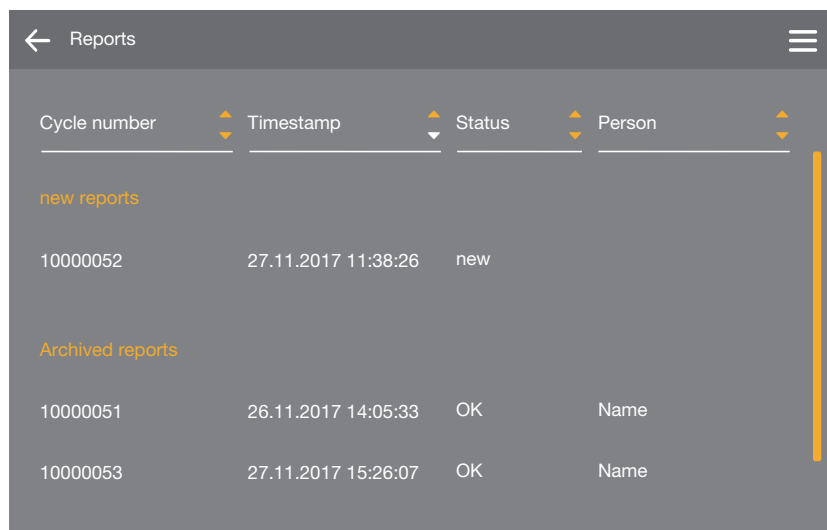Once you have correctly authenticated yourself, a PDF will be generated from the report and stored in the “Miele/DataDiary” folder on the tablet.

The report will be stored in the list of edited reports and the logged-in user will be stored as the editor (Person).

Cancelling reports

Report 05.12.2017 08:36:12

Final rinse 2 -----

09:40:14 Demineralised water 9.0l
 09:41:31 ACTUAL amount 9.0l
 09:41:51 ACTUAL conductivity 1µS/cm
 Max. conductivity 150µS/cm
 Conductivity OK

Programme cancelled
 Process parameters not achieved
 09:47:11 FINISH

Create cycle report?

☐ Process parameters OK
☐ Visual cycle check OK
☐ Report OK

Create failure report Create approvals report

Edit later

- If the report is not OK, tap the Create failure report button.

Report 05.12.2017 08:36:12

Final rinse 2 -----

09:40:14 Demineralised water 9.0l
 09:41:31 ACTUAL amount 9.0l
 09:41:51 ACTUAL conductivity 1µS/cm
 Max. conductivity 150µS/cm
 Conductivity OK

Programme cancelled
 Process parameters not achieved
 09:47:11 FINISH

Next steps?

☒ Repeat washing/disinfection
☐ Reject cycle
☐ -----

Back Confirm

Edit later

You must specify how the batch is to be processed. You can either choose one of the suggestions, or choose the third option and enter free text.

- Choose an option and tap the Confirm button.

← Report 05.12.2017 08:36:12

Final rinse 2 -----

09:40:14 Demineralised water 9.0l

09:41:31 ACTUAL amount 9.0l

09:41:51 ACTUAL conductivity 1µS/cm

Max. conductivity 150µS/cm

Conductivity OK

Programme cancelled

Process parameters not achieved

09:47:11 FINISH

Please authenticate yourself

Select user

Password

Show password

Authentication with NFC tag

Back Confirm

Edit later

You will then be prompted to authenticate yourself.

- Log in with your user name and password.

Or

- Log in using your NFC card by holding it against the NFC sensor in your tablet. The sensor is usually located on the back of the tablet.

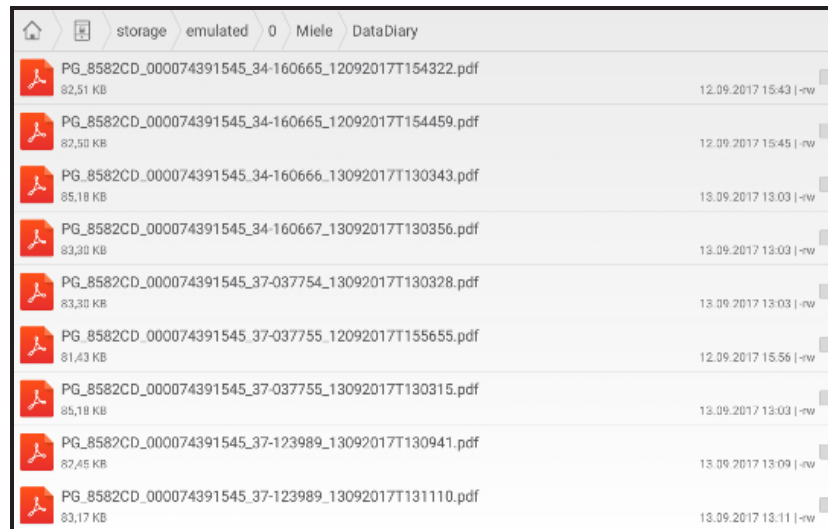
The report will be stored in the list of edited reports and the logged-in user will be stored as the editor (Person).

Aborting editing

Select **Edit later** to abort the process of editing the report. This will clear any ticks you have placed in boxes and anything you have entered in the comment area.

Archiving reports

A PDF file is automatically generated from any edited report and stored in the tablet's internal memory.

The PDF files are stored in the “Miele/DataDiary” folder.

⚠ The tablet's internal memory is not a suitable location for archiving reports permanently, however. For this reason, you should regularly transfer the reports to a storage medium with a backup solution.

Retrospective output of reports

If there are WiFi problems during data transmission or the tablet's battery dies, the batch reports can be transmitted retrospectively.

- To do this, you need to open the menu on your machine using the following path:
Further settings/Interface/RS232/Print reports

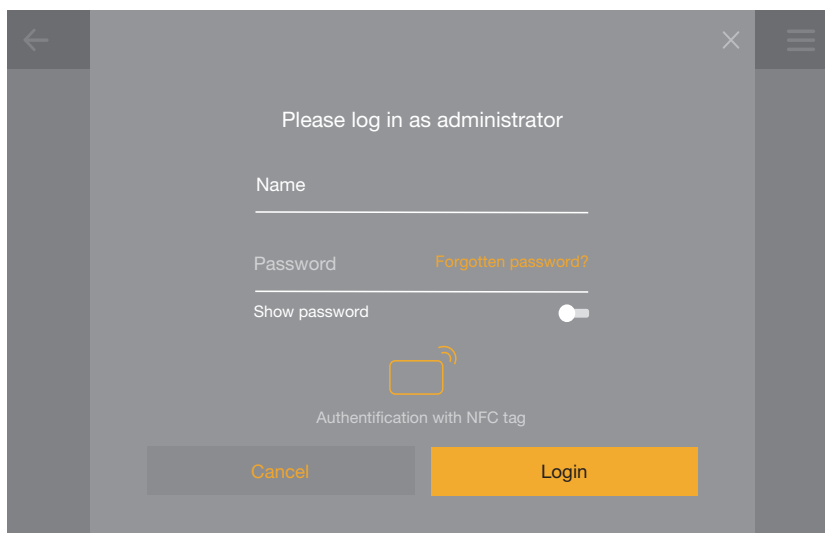
Further instructions and additional information can be found in “Process documentation” in the operating instructions for your machine.

Opening the Administration menu

Administration

Only the app administrator can open the Administration menu.

- Start the Miele DataDiary app by tapping the app icon.
- Open the menu by either tapping the icon in the top right-hand corner or swiping from the right-hand side of the screen towards the centre.
- Select the Administration option.

The screenshot shows a login dialog box with a grey background. At the top, it says "Please log in as administrator". Below this are two input fields: "Name" and "Password". To the right of the password field is a link that says "Forgotten password?". Below the password field is a "Show password" toggle switch, which is currently turned off. Below the toggle is an NFC symbol (a rectangle with a curved line) and the text "Authentication with NFC tag". At the bottom are two buttons: "Cancel" (grey) and "Login" (orange).

You will then be prompted to log in as the administrator.

- Enter the password.

Or

- Log in using your NFC card by holding it against the NFC sensor in your tablet. The sensor is usually located on the back of the tablet.

The Administration menu allows you to make basic settings, such as creating new users or setting up a WiFi connection.

Language

Here, you can choose the language for the Miele DataDiary app.

The language you select does not affect the language in which the batch reports are output.
The report language is set on your machine.
To do this, consult the information in the operating instructions for the XKM module and the machine.

- Select the Language option.

- Select a language by tapping the relevant field.

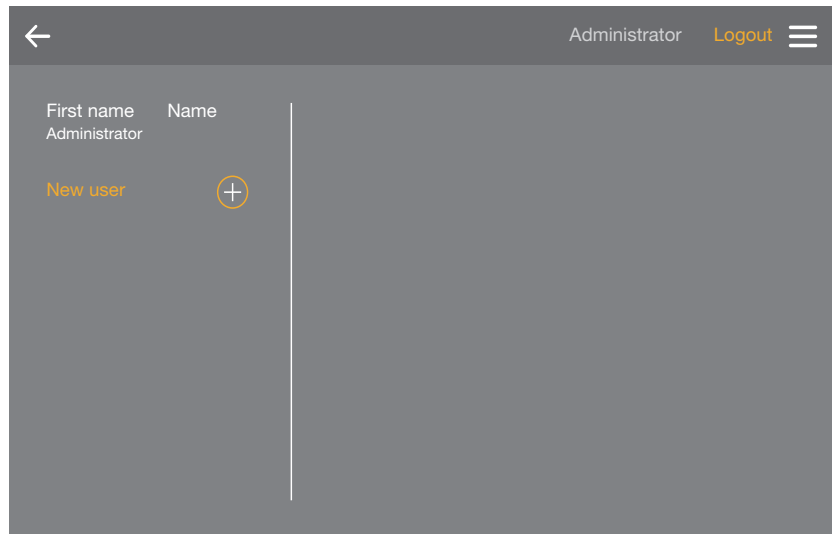
The language will change to this immediately.

- Exit the menu using the arrow icon <— at the top left.

Users

In this menu, you can create new users and edit or delete existing users.

- Tap the User option.


This will display all the registered users.

- To edit or delete user data, tap one of the users and press the relevant button. It is not possible to delete the administrator unless the administrator rights have been transferred to another user (Change administrator).

Or

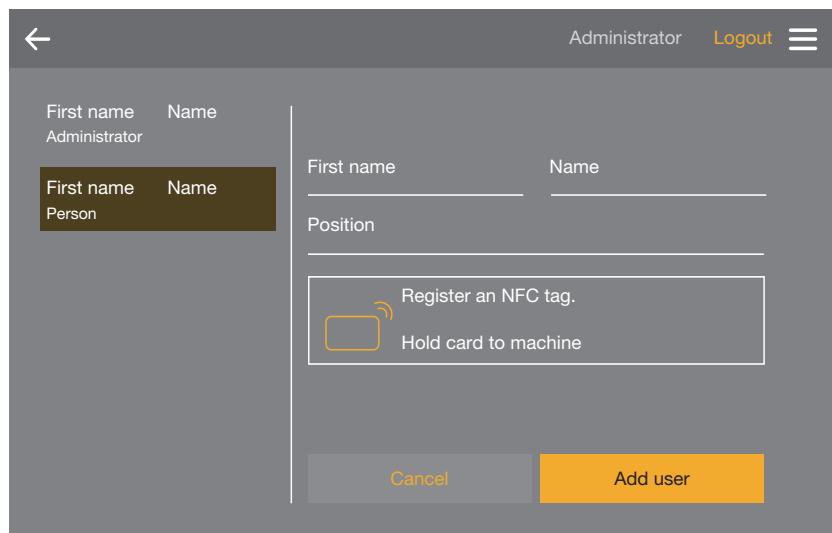- Tap the (+) icon to create a new user.

- Complete the First name, Name and Position fields, all of which are mandatory. When you want to complete a field, start by tapping it.

You then have the option of storing a chip card with an NFC (near-field communication) tag for the user.

If someone logs in using an NFC card, there is no need for them to enter their user name or password.

To use NFC cards, you need a tablet that supports NFC. 3 NFC cards are supplied with the XKM module. You can obtain additional cards from Miele.

- Hold the NFC card against the back of the tablet. In most cases, the NFC module that reads the NFC card is located at the edge. However, the exact position of the NFC antenna will vary from one tablet model to another.

If the message NFC tag stored: xxxxxxxx appears, where xxxxxxxx represents the ID number of the NFC card, this means that the card has been successfully assigned to the user account.

Use **Reset** to remove the NFC card assignment.

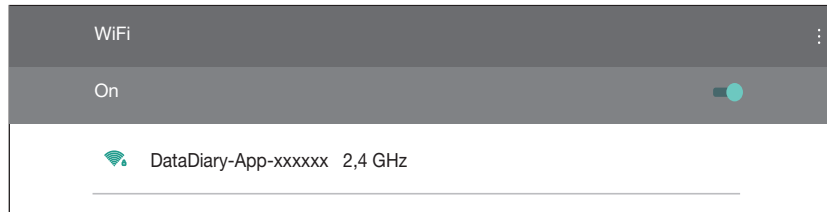
- Tap the **Add user** button.

New users create their own password the first time they log in.

en - Administration

WiFi configuration Consult the information in the operating instructions for the XKM 3100 WCDS interface module.


- Switch on the machine and make sure that the serial interface (RS232) on it is activated.
- Open the Administration menu.
- Tap the WiFi configuration option.

- Activate WiFi.

All available WiFi networks in the vicinity will then appear.

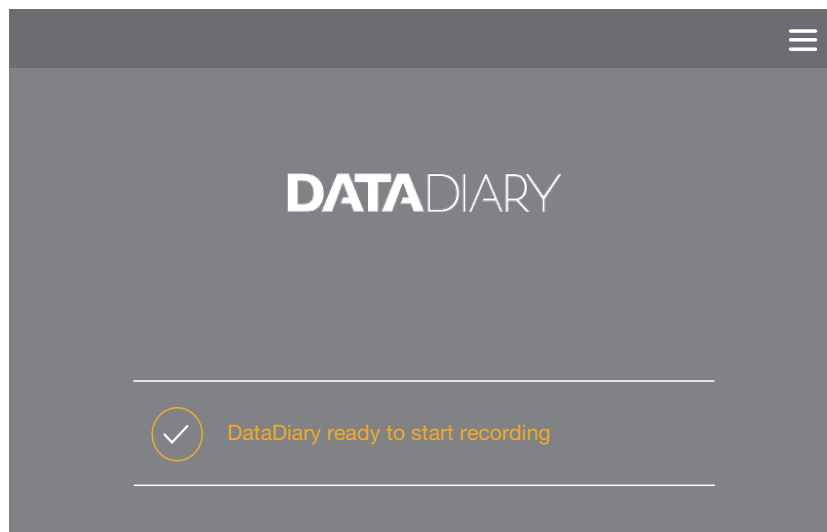- Select the XKM module's WiFi network and enter the password.
 - The WiFi SSID is DataDiary-AP-XXXXXX, where XXXXXX corresponds to the last 6 letters of the XKM module MAC address.
 - The WiFi password is the serial number of the XKM module without the initial zeroes.

As soon as a WiFi connection has been established, this will be displayed in the app.

XKM FW update

The Miele DataDiary app has an update wizard which you can use to update the XKM module firmware. This requires a separate WiFi network with an Internet connection.

- Select the XKM-FW update option.
- Enter the network name and the password.
- Tap Connect.

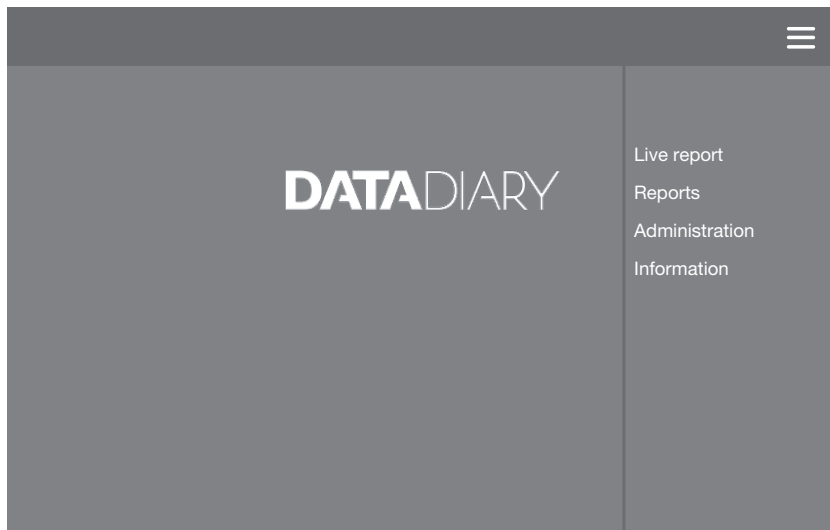
The WiFi connection between the tablet and the XKM module will now be disconnected.

- Follow the instructions in the wizard.

Information

Here, you can view information about the app.

- Open the menu by either tapping the icon in the top right-hand corner or swiping from the right-hand side of the screen towards the centre.

- Select the Information option.

The following information is provided here:

- Model designation of your machine
- WiFi SSID
- XKM module model
- IP address of the XKM module
- Miele DataDiary app version
- Legal notice
- Terms of use
- Information regarding data protection

Saving energy

Most tablets offer various options for saving energy, such as a sleep mode or energy-saving mode. These options reduce the performance of the tablet and restrict the background activities that apps are able to carry out. Depending on the tablet model and the operating system, using these options may abort the transmission of data between the Miele app and the Miele machine.

If you do wish to use energy-saving options on your tablet, please note the following:

- Make sure that the Miele app still remains active in the background if energy-saving mode is used.
- When configuring the WiFi, make sure that you set it to remain active when the tablet is in energy-saving mode.
- Make sure that the tablet battery has enough charge, or use a mains power supply. Some tablet models enter energy-saving mode automatically if their battery level is low. This may impair data transmission or cause it to abort completely.
- If you have set your tablet to shut down automatically at a specific time, make sure you start the programme in enough time for it to finish before the tablet shuts down.

For more information about configuring energy-saving mode and WiFi, consult the operating instructions for your tablet.

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Annuler un protocole

The screenshot shows a mobile application interface for a protocol. The top bar displays a back arrow, the title 'Protocole', the date '05.12.2017', and the time '08:36:12'. On the left, a log entry for 'Final rinse 2' is shown, including timestamps and conductivity readings. The status 'Programme cancelled' and 'Process parameters not achieved' are displayed, followed by '09:47:11 FINISH'. On the right, a dialog titled 'Créer le protocole de charge ?' contains three unchecked checkboxes: 'Paramètre de process ok', 'Contrôle visuel charge ok', and 'Protocole correct'. At the bottom of the dialog are two buttons: 'créer proto. annul.' and 'créer protoco. val.', and a 'Plus tard' link.

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The screenshot shows the same mobile application interface. The left log entry is identical. The right dialog is titled 'Comment procéder pour cette charge ?' and features three radio button options: 'Nouv. traitement et désinfection' (which is selected), 'Trier les charges', and an empty option with a text input field. At the bottom of the dialog are two buttons: 'Retour' and 'Confirmer', and a 'Plus tard' link.

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Programme cancelled
Process parameters not achieved
09:47:11 FINISH

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The screenshot displays a mobile application interface with a dark grey background. At the top, a header bar contains a back arrow, the text 'Protocollo', the date '05.12.2017', the time '08:36:12', and a hamburger menu icon. The main content is split into two panels. The left panel, titled 'Final rinse 2', shows a log of events: '09:40:14 Demineralised water 9.0l', '09:41:31 ACTUAL amount 9.0l', '09:41:51 ACTUAL conductivity 1µS/cm', 'Max. conductivity 150µS/cm', 'Conductivity OK', and '09:47:11 FINISH'. Below this, it states 'Programme cancelled' and 'Process parameters not achieved'. The right panel, titled 'Autenticarsi', contains a user selection dropdown, a password field, a password visibility toggle, and an NFC authentication icon with the text 'Autenticazione tramite tag NFC'. At the bottom of the right panel are three buttons: 'Indietro', 'Confermare', and 'Modificare più tardi'.

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Autorizar protocolos

The screenshot shows a mobile application interface for 'Protocolo'. The top bar displays a back arrow, the title 'Protocolo', the date '04.12.2017', and the time '07:24:46'. On the left, there is a list of items under 'Drying block' and 'Programme finished'. The main area on the right asks '¿Es posible crear el protocolo de ciclo?' and contains three checkboxes: 'Parámetro proceso OK' (checked), 'Control visual ciclo OK' (checked), and 'Protocolo OK' (unchecked). At the bottom, there are three buttons: 'Crear prot.' (disabled), 'Crear protocolo alta' (active), and 'Modificar más tarde'.

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
The screenshot shows the 'Autenticarse' screen. The top bar is identical to the previous one. The left sidebar remains the same. The main area contains a login form with fields for 'Seleccionar usuario' (with a dropdown arrow), 'Contraseña', and a 'Mostrar contraseña' toggle switch. Below the password field is an icon of a card with a signal wave, labeled 'Autenticación via tarjeta NFC'. At the bottom, there are three buttons: 'Atrás' (disabled), 'Confirmar' (active), and 'Modificar más tarde'.

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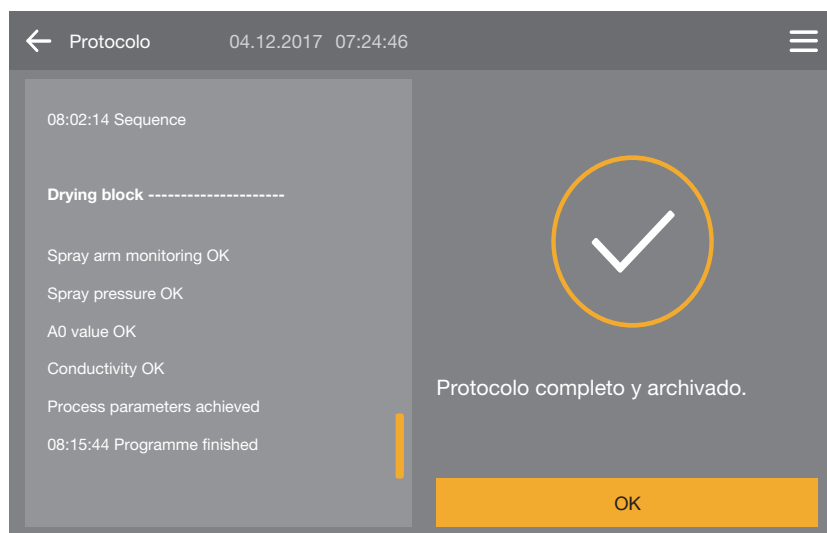This screenshot shows the same protocol summary on the left. The confirmation dialog on the right now asks '¿Siguientes pasos?'. It contains three radio button options: 'Nueva limpieza y desinfección' (selected), 'Eliminar ciclo', and an empty option with a text input field. At the bottom, there are two buttons: 'Atrás' (disabled) and 'Confirmar' (active), and a link 'Modificar más tarde'.

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The screenshot shows a mobile application interface with a dark grey background. At the top, there is a header bar with a back arrow, the word 'Protocol', the date '05.12.2017', the time '08:36:12', and a menu icon. The main content is split into two panels. The left panel, titled 'Final rinse 2', contains a log of events: '09:40:14 Demineralised water 9.0l', '09:41:31 ACTUAL amount 9.0l', '09:41:51 ACTUAL conductivity 1µS/cm', 'Max. conductivity 150µS/cm', 'Conductivity OK', '-----', 'Programme cancelled', 'Process parameters not achieved', and '09:47:11 FINISH'. The right panel, titled 'Authenticatie vereist', contains fields for 'Gebruiker kiezen' (with a dropdown arrow), 'Wachtwoord', and a toggle for 'Wachtwoord weergeven'. Below these is an NFC icon and the text 'Authenticeren met NFC-tag'. At the bottom of the right panel are three buttons: 'Terug' (grey), 'Bevestigen' (orange), and 'Later bewerken' (yellow text).

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
The screenshot shows the same 'Raport' screen, but now it's asking for authentication. The title is 'Proszę się uwierzytelnić' (Please authenticate). There are two input fields: 'Wybierz użytkownika' (Select user) with a dropdown arrow, and 'Hasło' (Password). Below the password field is a toggle switch labeled 'Pokaż hasło' (Show password). There is also an NFC icon and the text 'Uwierzytelnienie przez tag NFC' (Authentication by NFC tag). At the bottom are two buttons: 'Powrót' (Return) and 'Potwierdź' (Confirm), and a link 'Edytuj później' (Edit later).

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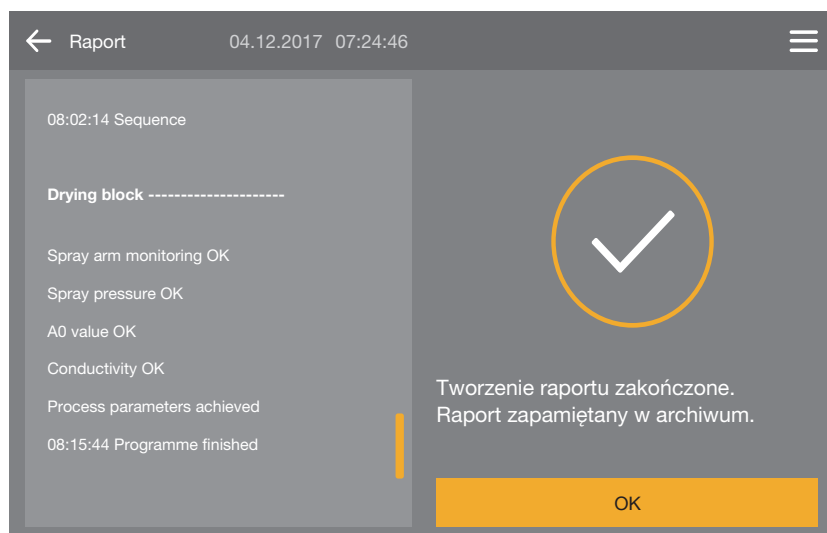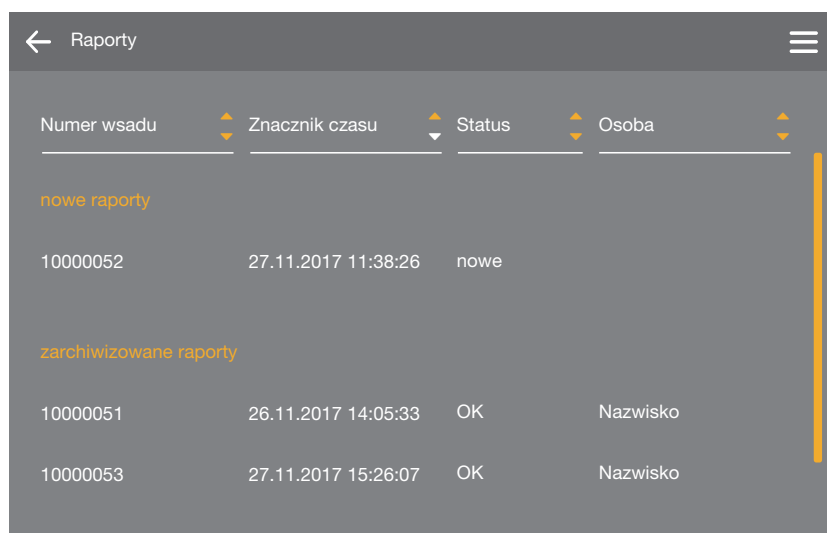
The screenshot shows a mobile application interface for a report. The top bar contains a back arrow, the title 'Raport', the date '05.12.2017', and the time '08:36:12'. On the left, a log entry for 'Final rinse 2' is shown with details: '09:40:14 Demineralised water 9.0l', '09:41:31 ACTUAL amount 9.0l', '09:41:51 ACTUAL conductivity 1µS/cm', 'Max. conductivity 150µS/cm', and 'Conductivity OK'. Below this, it states 'Programme cancelled' and 'Process parameters not achieved', followed by '09:47:11 FINISH'. On the right, a question asks 'Czy można utworzyć raport cykli programowych?' (Can I create a program cycle report?). There are three checkboxes: 'Parametry procesowe OK', 'Kontrola wizualna OK', and 'Raport OK'. At the bottom, there are two buttons: 'Utwórz raport usterek' (Create fault report) and 'Utwórz raport akceptacji' (Create acceptance report), and a link 'Edytuj później' (Edit later).

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The screenshot shows a mobile application interface with two main panels. The left panel, titled 'Raport', displays a report for 'Final rinse 2' with the following details: '09:40:14 Demineralised water 9.0l', '09:41:31 ACTUAL amount 9.0l', '09:41:51 ACTUAL conductivity 1µS/cm', 'Max. conductivity 150µS/cm', and 'Conductivity OK'. Below this, it states 'Programme cancelled', 'Process parameters not achieved', and '09:47:11 FINISH'. The right panel, titled 'Proszę się uwierzytelnić', is a login screen. It includes a dropdown menu for 'Wybierz użytkownika', a password field labeled 'Hasło', and a toggle switch for 'Pokaż hasło'. Below the password field is an NFC icon and the text 'Uwierzytelnienie przez tag NFC'. At the bottom of the right panel are three buttons: 'Powrót' (grey), 'Potwierdź' (orange), and 'Edytuj później' (yellow).

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
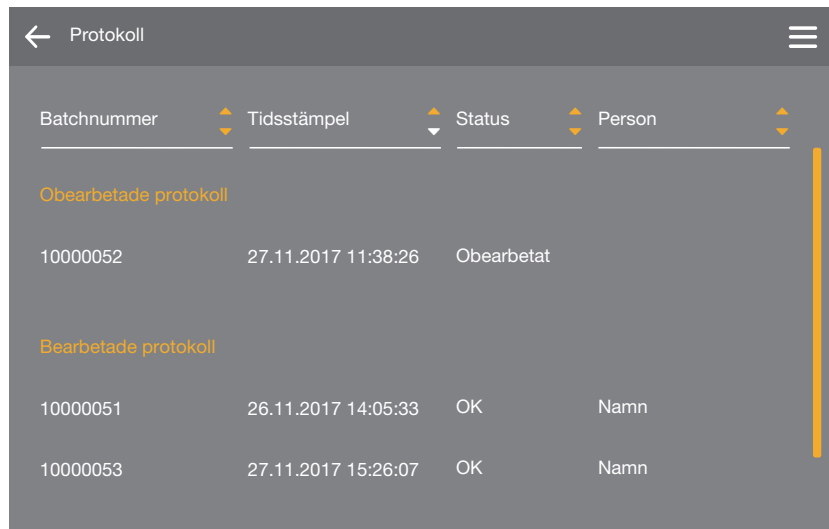
The screenshot shows a mobile application interface titled "Protokoll". At the top, there is a header bar with a back arrow, the title "Protokoll", and a menu icon. Below the header, there is a table with four columns: "Batchnummer", "Tidsstämpel", "Status", and "Person". The table is divided into two sections: "Obearbetade protokoll" (Unprocessed protocols) and "Bearbetade protokoll" (Processed protocols). The "Obearbetade protokoll" section contains one row with the following data: Batchnummer: 10000052, Tidsstämpel: 27.11.2017 11:38:26, Status: Obearbetat. The "Bearbetade protokoll" section contains two rows with the following data: Row 1: Batchnummer: 10000051, Tidsstämpel: 26.11.2017 14:05:33, Status: OK, Person: Namn; Row 2: Batchnummer: 10000053, Tidsstämpel: 27.11.2017 15:26:07, Status: OK, Person: Namn.

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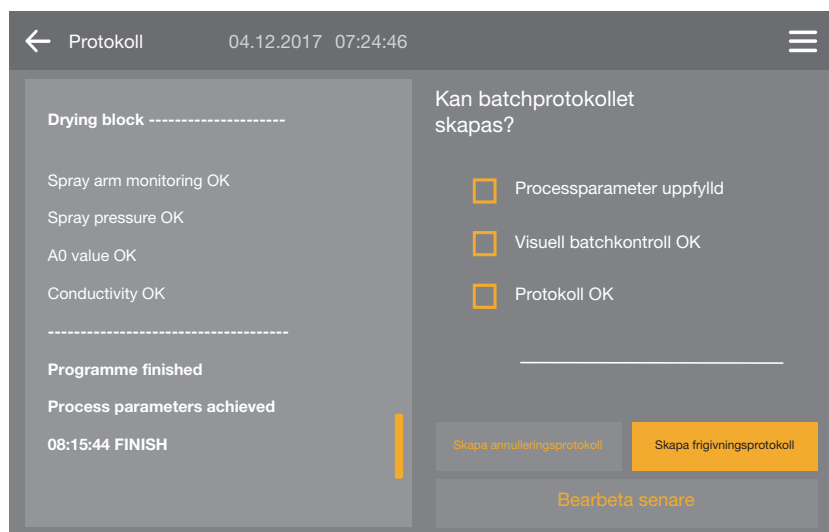The screenshot shows a mobile application interface titled "Protokoll". At the top, there is a header bar with a back arrow, the title "Protokoll", and a menu icon. Below the header, there is a timestamp "04.12.2017 07:24:46". The main content area is divided into two sections. The left section, titled "Drying block", contains a list of status items: "Spray arm monitoring OK", "Spray pressure OK", "A0 value OK", and "Conductivity OK". Below this list, there is a section titled "Programme finished" with the text "Process parameters achieved" and a timestamp "08:15:44 FINISH". The right section, titled "Kan batchprotokollet skapas?" (Can the batch protocol be created?), contains three checkboxes: "Processparameter uppfyllt", "Visuell batchkontroll OK", and "Protokoll OK". Below these checkboxes, there are two buttons: "Skapa anslutningsprotokoll" (Create connection protocol) and "Skapa frigivningsprotokoll" (Create release protocol). At the bottom of the right section, there is a button labeled "Bearbeta senare" (Process later).

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Frisläppa protokoll

The screenshot shows the 'Protokoll' screen with a header bar containing a back arrow, the title 'Protokoll', the date and time '04.12.2017 07:24:46', and a menu icon. The main content is divided into two panels. The left panel, titled 'Drying block -----', lists the following status items: 'Spray arm monitoring OK', 'Spray pressure OK', 'A0 value OK', and 'Conductivity OK'. Below these, it states 'Programme finished', 'Process parameters achieved', and '08:15:44 FINISH'. The right panel, titled 'Kan batchprotokollet skapas?', contains three checkboxes: 'Processparameter uppfylld' (checked), 'Visuell batchkontroll OK' (checked), and 'Protokoll OK' (unchecked). At the bottom of the right panel, there are three buttons: 'Skapa annulleringsprotokoll' (grey), 'Skapa frigivningsprotokoll' (orange), and 'Bearbeta senare' (grey).

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The screenshot shows the 'Protokoll' screen with the same header bar as the previous image. The main content is divided into two panels. The left panel is identical to the previous one, showing the 'Drying block' status. The right panel, titled 'Identifiera dig', contains a login form with the following fields: 'Välja användare' (with a dropdown arrow), 'Lösenord', and 'Visa lösenord' (with a toggle switch). Below the password field, there is an icon of an NFC tag and the text 'Identifiering via NFC-tag'. At the bottom of the right panel, there are three buttons: 'Tillbaka' (grey), 'Bekräfta' (orange), and 'Bearbeta senare' (grey).

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