# **Sonosite PX**

## **User Guide Errata**

---

**Manufacturer**

FUJIFILM SonoSite, Inc.

2 1919 30th Drive SE

Bothell, WA 98021 USA

T: +1-888-482-9449 or +1-425-951-1200

F: +1-425-951-1201

**EC Authorized Representative**

FUJIFILM SonoSite B.V.

Joop Geesinkweg 140

1114 AB Amsterdam,

The Netherlands

**Australia Sponsor**

FUJIFILM SonoSite Australasia Pty Ltd

114 Old Pittwater Road

BROOKVALE, NSW, 2100

Australia

**Caution** Federal (United States) law restricts this device to sale by or on the order of a physician.

SONOSITE and the SONOSITE logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P29198-02

Publication date: October 2020

Copyright © 2020 FUJIFILM SonoSite, Inc. All rights reserved.

# Sonosite PX User Guide Errata

<b>Introduction</b> .....	<b>1</b>
Getting help.....	1
<b>Getting started</b> .....	<b>2</b>
Power and battery indicators.....	2
Clinical monitor.....	2
<b>Scanning</b> .....	<b>4</b>
Needle guide control.....	4
<b>Measurement references</b> .....	<b>4</b>
<b>Cleaning and disinfecting</b> .....	<b>5</b>
<b>Safety</b> .....	<b>5</b>
Specifications.....	5

## Introduction

The content in this document corrects the Sonosite PX system user guide; the revision will be made in the next update.

## Getting help

For technical support, please contact FUJIFILM Sonosite as follows:

<b>Phone (U.S. or Canada)</b>	+1 (877) 657-8118
<b>Phone (Outside U.S. or Canada)</b>	+1 (425) 951-1330, or call your local representative
<b>Fax</b>	+1 (425) 951-6700
<b>E-mail</b>	ffss-service@fujifilm.com
<b>Web</b>	www.sonosite.com
<b>Europe Service Center</b>	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
<b>Asia Service Center</b>	+61 2 9938 8700 ffss-apacme-service@fujifilm.com

Printed in the U.S.

# Getting started

## Power and battery indicators

The behavior of the lights on the touch panel and the system lid has been corrected.

**Table 1: Power and battery indicators**

Indicator	Status
Lights on the touch panel and the lid	The lights turn on when the system is off or asleep. A solid light means that the system is connected to AC power and the batteries are charged. A blinking light means that the batteries are charging. A pattern of three blinks indicates that the battery charge is greater than 70%. Two blinks indicates that the charge is between 40% and 70%. One blink indicates that the battery charge is between 10% and 40%.

## Clinical monitor

The clinical monitor figure has been corrected (see [page 3](#)).

The clinical monitor displays the ultrasound image as well as details about the exam and system status. The clinical monitor is not a touchscreen.

**Figure 1 Clinical monitor layout**

- |   |                                  |    |  |
|---|----------------------------------|----|--|
| 1 | Measurement and calculation area | 6  | Clinician, Department, and Hospital                  |
| 2 | Patient header                   | 7  | Selected transducer, exam type, and MI and TI values |
| 3 | Ultrasound image                 | 8  | Imaging mode or modes, selected controls             |
| 4 | Orientation marker               | 9  | System status area                                   |
| 5 | Date and time                    | 10 | Depth scale  |

# Scanning

## Needle guide control

The following table has been corrected.

**Table 2: Needle guide control availability**

Available with	Not available with
IC10-3 (fixed-angle)	variable angle needle brackets
L19-5 (transverse-angle)	a reduced field of view
	the centerline feature

## Measurement references

The following reference has been corrected.

### Left Ventricular mass in gm for 2D

Lang, R., Badano, L., et al. "Recommendations for Cardiac Chamber Quantification by Echocardiography in Adults: An Update from the American Society of Echocardiography and the European Association of Cardiovascular Imaging." *J Am Soc Echocardiogr.* (2015), 28: p.1-39

$$LV \text{ mass} = 1.05 * \{[(5/6) * A_1 * (a + d + t)] - [(5/6) * A_2 * (a + d)]\}$$

where:

$A_1$  = short axis area, diastole (Epi)

$A_2$  = short axis area, diastole (Endo)

a = distance from the minor axis to the endocardium at the LV apex

d = distance from the minor axis to the mitral valve plane

a + d = LV length from mitral valve plane to endocardium at LV apex (Apical)

t = mean wall thickness =  $\sqrt{(Epi / \pi)} - \sqrt{(Endo / \pi)}$

## Cleaning and disinfecting

The following table and link have been corrected.

**Table 3: High-level disinfectants compatible with Sonosite PX transducers:**

Product <sup>a, b</sup>	Compatible transducers
Cidex	C5-1, IC10-3, L12-3, L15-4, L19-5, P5-1
Cidex OPA	C5-1, IC10-3, L12-3, L15-4, L19-5, P5-1
Revital-Ox RESERT	C5-1, IC10-3, L12-3, L15-4, L19-5, P5-1

a. Refer to the manufacturer's instructions for concentration, temperature, and duration.

b. For a complete list of approved cleaners and disinfectants, refer to the cleaners and disinfectants tool available at [www.sonosite.com/support/cleaners-disinfectants](http://www.sonosite.com/support/cleaners-disinfectants).

## Safety

The following have been corrected.

## Specifications

### System

- ▶ **Length:** 15.45 in. (39.25 cm)
- ▶ **Width:** 13.32 in. (33.84 cm)
- ▶ **Height:** 14.60 in. (37.09 cm)

### Environmental limits

The following table has been corrected.

**Table 4: Shipping and storage limits**

Limit type	System, transducers, and stand	System batteries
Temperature	-35–65°C (-31–149°F)	-20–60°C (-4–140°F) for 1 month -20–45°C (-4–113°F) for 3 months -20–23°C (-4–73°F) for 1 year
Humidity	15–95% R.H.	15–95% R.H.
Atmospheric pressure	500–1060 hPa (0.5–1.05 ATM)	500–1060 hPa (0.5–1.05 ATM)

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